

Fabrication

IMPRESSION KIT

by point designs



point designs

POINT DESIGNS FABRICATION IMPRESSION KIT

Impression Requirements

- Step by step instructions are found in the 4 minute video on our website (or use QR code to the right)
- Store silicone in freezer until ready to use
- Make sure coating of Vaseline is sufficient to mat down hairs
- Ensure impression covers the entire hand including the styloids (no need to go more proximal)
- Ensure there are no thin spots in the impression; use second tub of silicone as needed
- Make sure hand is relaxed with the thumb midway between an opposed and lateral position
- Send unfilled impression to our fabrication lab

Impression Kit Materials

Tounge Depressor (x2)

Silicone Impression
Material (x2)


Silicone Catalyst (x2)

Petroleum Jelly (x1)



point designs

Impression How To Video

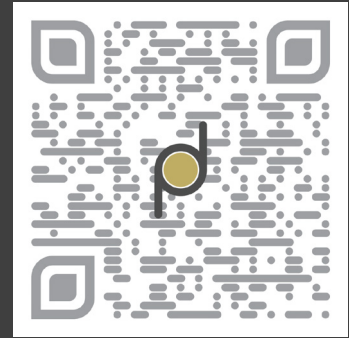

Schedule a Consultation

